

Mediven Flight Socks Information

Class 1 compression hose used in the Lancet survey.

Measuring Guide

You need to take the measurement around the leg just above the left ankle when ordering.

CM	Size	Inches
18-20	S	7 ¹ / ₄ -8
20-23	M	8-9
23-26	L	9-10 ¹ / ₄
26-29	XL	10 ¹ / ₄ -11 ¹ / ₂
29-32	XXL	11 ¹ / ₂ -12 ¹ / ₂

These stockings have a compression level between 18.4 – 21.1 mmhg and are clinically proven.

Instructions for Putting on Mediven Travel

We advise that you put on your Mediven travel stockings prior to your arrival at the airport. Before pulling the stocking on turn it "inside out" right down to the foot. This helps you slip easily into the foot section. Please check the correct fit at the heel. Smooth the leg section up evenly up your leg, taking care not to twist the stocking, pull on the edges or overstretch it. If you pulled the stocking up too far, ease it back down to your ankle and then pull it up again with less stretch.

To Order

Please telephone 01234 356980, order on-line, or complete the order form on the rear of this sheet and fax it to 01234 340066 or post it to us at the address below.

<http://www.bostockhealthcare.co.uk/ccp6/gbu0-catshow/Flight-Socks.html>